CIDEZYME®

Enzymatic Detergent

REF 2258 / REF 2260

US GB IE CA

INSTRUCTIONS FOR USE

1. Do not add bleach or other chemicals.

Add 8 ml of CIDEZYME® Enzymatic Detergent liquid concentrate per 1 liter of water. For equipment with dried-on organic matter, use 16 ml per 1 liter of water and/or use warm water.

CIDEZYME Detergent may be used for precleaning instruments in automated equipment such as endoscope reprocessors, ultrasonic cleaners and washer disinfectors, in accordance with equipment manufacturers' recommendations.

2. Soak equipment immediately after use, until all organic matter is dissolved and removed. A minimum soak of 1 minute is recommended. Extend soak time for equipment with dried-on organic matter. Flush solution through all channels.

Please note: Soak time for Aluminum no greater than 8 hours.

3. If necessary, mechanically clean equipment with brushes or soft cloths as recommended by the equipment manufacturer.

4. Thoroughly rinse the equipment, aspirating water through all channels to remove traces of CIDEZYME Detergent.

5. Rough dry equipment, including channels. Equipment is now ready for treatment with a high level disinfectant or sporicidal solution.

6. Discard used solution after each use. Flush used solution down the drain with water. CIDEZYME Detergent is not intended as a solution for holding or storing equipment.

7. Store at controlled room temperature 15°-30°C (59°-86°F).

Thoroughly rinse container before disposing. Use by expiration date on immediate container.

RISK/SAFETY INFORMATION:

Danger

Causes serious eye irritation. May cause allergy or asthma symptoms or breathing difficulties if inhaled. Obtain special instructions before use. Avoid breathing dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection.

Wear respiratory protection. If INHALED: Remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: Call a doctor. If eye irritation persists: Get medical advice/attention.

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Manufactured for:

ADVANCED STERILIZATION PRODUCTS

Division of Ethicon, Inc.
a Johnson & Johnson company

33 Technology Drive, Irvine, CA 92618 • 1-888-783-7723

For technical information please contact your ASP representative
24-Hour Number: 1-800-451-8346 or 1-760-602-8700

EC REP Johnson & Johnson MEDICAL GmbH
Robert-Koch-Str. 1, 22851 Norderstedt, Germany

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