

GAMMEX® Non-Latex Sensitive

CHEMOTHERAPY PERMEATION REPORT

CHEMOTHERAPY DRUG	MINIMUM BREAKTHROUGH DETECTION TIME (MINUTES)	CHEMOTHERAPY DRUG	MINIMUM BREAKTHROUGH DETECTION TIME (MINUTES)
	ASTM D6978		ASTM D6978
Bleomycin/Blenoxane (15.0 mg/ml)	>240	Idarubicin (1.0 mg/ml)	>240
Busulfan (6.0 mg/ml)	>240	Ifosfamide (50.0 mg/ml)	>240
Carboplatin/Paraplatin (10.0 mg/ml)	>240	Irinotecan (20.0 mg/ml)	>240
Carmustine (3.3 mg/ml)	30.3	Mechlorethamine HCl (1.0 mg/ml)	>240
Cyclophosphamide (20.0 mg/ml)	>240	Melphalan (5.0 mg/ml)	>240
Cytarabine (100 mg/ml)	>240	Methotrexate (25.0 mg/ml)	>240
Daunorubicin (5.0 mg/ml)	>240	Mitomycin C (0.5 mg/ml)	>240
Docetaxel (10.0 mg/ml)	>240	Mitoxantrone (2.0 mg/ml)	>240
Doxorubicin Hydrochloride (2.0 mg/ml)	>240	Oxaliplatin (2.0 mg/ml)	>240
Etoposide (20.0 mg/ml)	>240	Paclitaxel (6.0 mg/ml)	>240
Fludarabine (25.0 mg/ml)	>240	ThioTEPA (10.0 mg/ml)	61.0
Fluorouracil (50.0 mg/ml)	>240	Vincristine Sulfate (1.0 mg/ml)	>240
Gemcitabine (38.0 mg/ml)	>240		

RATING SYSTEM

RECOMMENDED

The results for this specific chemotherapy drug suggest that the glove would provide an adequate barrier for use in most applications. Breakthrough occurs in >60 minutes.


CAUTION

The results require additional consideration to determine suitability for use. Breakthrough occurs in >10 to 60 minutes.

NOT RECOMMENDED

Not recommended for use. Breakthrough occurs in ≤10 minutes.

IMPORTANT: ASTM D6978 testing performed by a third-party accredited laboratory. Stated breakthrough times were determined under laboratory conditions that may not reflect actual usage. Variation in the environment or a mix of chemotherapy drugs used may impact breakthrough times. Users should test the suitability of this product against their specific chemotherapy drugs and environment.

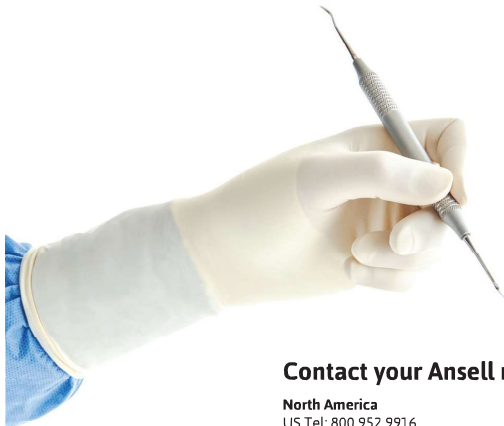

GAMMEX® Innovation that Protects

Driven by innovation, GAMMEX® surgical gloves are engineered to respond to the evolving user needs of the surgical community. Featuring unique formulations and cutting-edge technologies, ergonomic comfort and superior protection, healthcare workers around the world entrust their hands to ours.

Contact your Ansell representative for ordering or more information.

North America

US Tel: 800 952 9916
CA Tel: 1-844-494-7854
Email: insidesalesus@ansell.com

Central & South America

Tel: +52(442) 296 20 50
Email: cslac@ansell.com
Brazil
Tel: +55-11-3356-3100
Email: luvas.medicas@ansell.com

Europe, Middle East & Africa

Tel: +32 (0) 2 528 74 00
Email: info@ansell.eu

Asia Pacific

Tel: +603 8310 6688
Email: apac.medical@ansell.com

Australia & New Zealand

Tel: +61 3 9270 7270
Email: protection@ap.ansell.com