Pátek, 6. října 2023

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Friday, October 06, 2023

At O&M Halyard, we seek to constantly improve upon the quality of healthcare products and services available in the marketplace. We strive to deliver clinically-superior products with remarkable service to improve the wellbeing of the people we touch every day. In accordance with this vision, we want to inform our customers of additional chemical and chemotherapy drug testing performed on our **SKYBREEZE* Nitrile Powder-Free Exam Gloves**. In the US, the Food and Drug Administration requires submission and clearance of a premarket notification for medical exam gloves that are intended for use with chemotherapy drugs and requires glove products to be labeled with drugs tested and breakthrough times. Package labeling for Halyard exam gloves sold in the US meet this labeling requirement.

Although additional chemotherapy drug and chemical testing is not required, Halyard believes the more informed our customers are, the better they will be able to understand the appropriate use of our products. The following chemotherapy drugs and chemicals have been tested according to EN 374-1 (EN 16523 where applicable), ASTM D6978 (chemotherapy drugs) and ASTM F739 (chemicals). This data is provided for your information but is not intended to expand the intended use of the products beyond the scope of what has been cleared by the FDA or covered by the safety and performance requirements of the CE certification and the European harmonized standards.

If you have any questions, please do not hesitate to contact your Halyard qualified sales representative for more information.

Chemotherapy Testing Information - Permeation testing per ASTM D6978-05

| Chemotherapy Drugs Tested | Breakthrough time (minutes) SKYBREEZE* Nitrile |
|-------------------------------|--|
| Cisplatin (1.0 mg/ml) | No breakthrough up to 240 min |
| Cyclophosphamide(20 mg/ml) | No breakthrough up to 240 min |
| Dacarbazine (10 mg/ml) | No breakthrough up to 240 min |
| Doxorubicin HCL (2 mg/ml) | No breakthrough up to 240 min |
| Etoposide (20 mg/ml) | No breakthrough up to 240 min |
| 5-Fluorouracil (50 mg/ml) | No breakthrough up to 240 min |
| Methotrexate (25 mg/ml) | No breakthrough up to 240 min |
| Mitcomycin C (0.5 mg/ml) | No breakthrough up to 240 min |
| Paclitaxel (6 mg/ml) | No breakthrough up to 240 min |
| Vincristine Sulfate (1 mg/ml) | No breakthrough up to 240 min |
| Carmustine (3.3 mg/ml) | Do not use |
| ThioTEPA (10 mg/ml) | Do not use |

Chemical Testing Information- ASTM F739/EN 16523-1 (where applicable)

| Chemicals Tested | CAS # | Test Method | Breakthrough time (minutes) SKYBREEZE* Nitrile |
|--|------------|-------------|--|
| Acetone | 67-64-1 | EN 374-3 | <1 |
| Acrylamide (40%) | 79-06-1 | ASTM F739 | 33.3 |
| CaviCide | | ASTM F739 | < 10 |
| Chlorhexidine digluconate (4%) | 18472-51-0 | ASTM F739 | > 480 |
| Cidex OPA | | ASTM F739 | 6.7 |
| Citric acid | 77-92-9 | EN 374-3 | >480 |
| Ethyl Acetate | 141-78-6 | EN 374-3 | <1 |
| Ethanol (70%) | 64-17-5 | EN 16523-1 | 11 |
| Formalin, 10% Buffered (tested for Formaldehyde) | 50-00-0 | ASTM F739 | > 480 |
| Gluteraldehyde 50% | 111-30-80 | ASTM F739 | 20 |
| Hexane | 110-54-3 | EN 374-3 | 7 |
| Hydrochloric Acid, 37% | 7647-01-0 | ASTM F739 | > 40 |
| Hydrogen Peroxide, 30% | 7722-84-1 | EN 16523-1 | 39 |
| Isopropyl Alcohol, 70% | 67-63-0 | ASTM F739 | 8.0 |
| Methanol | 67-56-1 | EN 374-3 | <1 |
| MetriCide 14-Day | | ASTM F739 | 30 |
| OxyCide Concentrate | | ASTM F739 | 6.7 |
| Quaternary Cleaner | | ASTM F739 | > 480 |

Halyard gloves are tested by accredited independent laboratories: Akron Rubber & Development Lab (US), SGS, TUV

*Registered Trademarks or Trademarks of O&M Halyard or its affiliates © 2023

| Chemicals Tested | CAS # | Test Method | Breakthrough time (minutes) SKYBREEZE* Nitrile |
|------------------------------|-----------|--------------|--|
| Single Enzyme Solution | | ASTM F739 | 160 |
| Sodium Hydroxide (40%) | 1310-73-2 | EN 16523-1 | > 480 |
| Sodium Hydroxide (50%) | 1310-73-2 | ASTM F739 | > 480 |
| Sodium Hypochlorite (10-13%) | 7681-52-9 | ASTM F739 | > 480 |
| Sulfuric Acid (50%) | 7664-93-9 | ASTM F739 | > 480 |
| Sulfuric Acid (96%) | 7664-93-9 | EN 374-3 | 3 |
| n-Heptane | 142-85-5 | EN 16523-1 | 13 |
| Acetic acid (99%) | 64-19-7 | ASTM F739 | 0 |
| Ammonium Hydroxide (30%) | 1336-21-6 | ASTM F739 | < 6 |
| Hydrogen Peroxide (30%) | 7722-84-1 | EN 16523-1 | > 30 |
| Formaldehyde (37%) | 50-00-0 | EN 16523-1 | > 60 |
| Bone Cement | | ASTM F739-20 | Permeated – Do not use |
| N-Propanol (≥ 99.5%) | 71-23-8 | ASTM F739-20 | 23.4 |